LC148/d-11/2021

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ELECTRONIC SCALE WITH BODY FAT %, WATER %, MUSCLE MASS % (SKELETAL MUSCLE %) AND BASAL METABOLISM INDICATOR – INSTRUCTIONS AND WARRANTY

Dear customer, Laica would like to thank you for choosing one of our products, designed according to strict performance and quality criteria in order to ensure total satisfaction.

IMPORTANT
READ CAREFULLY BEFORE USE
KEEP IN A SAFE PLACE FOR FUTURE REFERENCE

The instruction manual must be considered an integral part of the product and must be kept for its entire life. If the product is transferred to another owner, its documentation must also be transferred in its entirety.

To ensure the safe and correct use of the product, the user must carefully read the instructions and warnings contained in the manual insofar as they provide important information regarding safety, user and maintenance instructions.

Should the instruction manual get mislaid or you require additional information or clarifications, please fill in the relevant form on the website: <https://www.laica.it/> in the FAQ and Support section.

Electronic scale that measures weight and calculates body composition (fat, water, and skeletal muscle percentages) and basal metabolism based on height, age, and gender.

INDEX

SYMBOLS KEY	page 8
SAFETY WARNINGS	page 8
BODY COMPOSITION	page 9
PRODUCT DESCRIPTION	page 9
INSTRUCTIONS FOR USE	page 10
MAINTENANCE	page 10
TROUBLESHOOTING	page 11
DISPOSAL PROCEDURE	page 11
WARRANTY	page 11

SYMBOLS KEY

-  Warning
-  Prohibition

SAFETY WARNINGS

- Before using, make sure the device is intact with no visible damage. If you have any doubts, do not use the device and contact the retailer.
- Keep the plastic bag used for packaging out of reach of children: suffocation hazard.
- This device must only be used for its intended purpose and in the manner indicated in the instructions.

- All other uses are considered improper and therefore hazardous. The manufacturer cannot be considered responsible for any damage due to improper or incorrect use.
- This device can be used by children on their own from the age of 8 upwards.
 - Individuals with reduced physical, sensory or mental capacities or unskilled people must only use the product if previously instructed on its use, in safety and only if informed of the dangers connected to the product itself. Children must not play with the device. Cleaning and maintenance operations must not be performed by children without adult supervision.
 - In the event of a failure and/or malfunction, do not tamper with the device. Always contact your retailer for repairs.
 - Handle the scale with care, protect it against knocks, extreme temperature fluctuations, humidity, dust, direct sunlight and sources of heat.
 - ⊘ Do NOT allow liquids to penetrate into the body of the scale. This appliance is not waterproof. Do NOT leave and do NOT use the scale in water-saturated environments; humidity rates greater than 85% or contact with water or other liquids may cause corrosion or compromise proper scale operation.
 - ⊘ Do NOT step onto the scale with wet feet, or if the surface of the scale is wet: slipping hazard.

	ATTENTION! BEFORE USING THIS APPLIANCE
---	--

- **The body fat, water, and skeletal muscle percentages and the basal metabolism calculated by the scale must only be considered to be indicative data. For further information, consult a doctor or dietician.**
- **Through the electrodes located on the platform, the scale sends a weak, low intensity electrical current that passes through the body. The scale is not dangerous in any way.**
- **The scale is not recommended for the following individuals as the data detected by the scale may not be accurate:**
 - o individuals wearing a pacemaker or other electronic medical device
 - o people with fever
 - o children under 10 years of age
 - o pregnant women
 - o individuals undergoing dialysis, with oedema in the legs or osteoporosis
 - o people who take cardiovascular medication

SAFE USE OF BATTERIES

- Remove the batteries if the scale is not used for long periods of time and keep them in a well-ventilated and dry place, at room temperature.
- ⊘ Do NOT recharge the batteries if they are not rechargeable.
- ⊘ Do NOT recharge rechargeable batteries using methods other than those indicated in the manual or using unsuitable equipment.
- ⊘ NEVER expose the batteries to sources of heat or direct sunlight. Failure to observe these instructions may result in damage to and/or the explosion of the batteries.
- ⊘ Do NOT throw the batteries into an open flame.
- The removal or replacement of batteries must always be performed by an adult.
- Keep the batteries far from the reach of children: swallowing batteries carries the risk of death. If batteries are swallowed, immediately contact a doctor.
- The acid contained in the batteries is corrosive. Avoid contact with the skin, eyes or garments

BODY COMPOSITION

The body composition is made up of:

- lean mass: all components without fat (water, muscle, connective tissue, and other non-fatty organic components), bone, muscles, bowels, water, organs, and blood;
- fatty mass: all fats that can be extracted from adipose tissue and other tissues/organs;
- bone mass.

Through the electrodes located on the platform, the scale sends a weak, low intensity electrical current that passes through the body. The different masses respond differently to the current: lean mass provides less resistance than the opposite.

This body resistance is called bioelectrical impedance. The relative percentage of the various body masses varies based on gender, age, and physical stature.

Body fat %

The body fat percentage indicated in the table below represents the total fat that is calculated by adding essential fat and storage fat.

Essential fat is necessary for various bodily functions.

The percentage is 3% in men and 12% in women.

Storage fat gives energy, protects against physical trauma, and insulates the body to preserve heat.

These percentages vary with the age and gender of the individual as shown in the table below.

If the percentages indicated by the scale do not fall within the values indicated as "normal" in the table, we recommend consulting with your family physician.

BODYFAT TABLE					
	Age	UNDERWEIGHT	NORMAL	OVERWEIGHT	OBESE
WOMEN	10-39	14-19%	19.1-29%	29.1-34%	> 34.1%
	40-55	15-20%	20.1-30%	30.1-35%	> 35.1%
	56-80	16-21%	21.1-31%	31.1-36%	> 36.1%
MEN	10-39	11-19%	19.1-24%	24.1-29%	> 29.1%
	40-55	12-20%	20.1-25%	25.1-30%	> 30.1%
	56-80	13-21%	21.1-26%	26.1-31%	> 31.1%

% of muscle mass (% of skeletal muscle)

There are three different types of muscle: smooth, striated skeletal, and striated cardiac.

Smooth muscle is made of smooth fibres. These are found in the walls of the digestive tract, respiratory tract and genital tract and in the wall of the venous and arterial blood vessels.

Human striated skeletal muscle is in the skeletal and facial muscles.

Cardiac striated muscle is found in the heart. This scale calculates the % of skeletal muscle and smooth muscle tissue in the body. On average, 40% of body mass is made up of skeletal muscle. Men usually have more pronounced muscles than women and the percentage may increase in those who practice sports. The percentage of muscle starts to decrease with age.

Water %

The % of water in the body differs based on gender: in healthy adult women it ranges from 45% to 60%

while in men it ranges from 50% to 65%.

The percentage of water is greater in childhood and decreases with age.

Body weight variations can occur in a short period of time (hours), a medium period of time (days) or over a long period (weeks).

Small variations in weight over a few hours is normal and due to water retention since extracellular water (which includes the interstitial fluid, plasma, lymph and transcellular fluid) is the only part capable of varying so rapidly.

When weight losses occur in just a few days, it is very likely that it is part of the total water.

Basal metabolic rate

The Basal Metabolic Rate, or BMR, is the minimum calories required for the human body to survive, in order to carry out basic functions such as circulation, breathing, digestion, and maintaining body temperature. BMR is calculated on a person when at physical and mental rest, lying down on their back, having not eaten for 12 hours, after at least 8 hours' sleep and with a constant body temperature of 20°C. Adult BMR varies from 1200 to 1800 kcal.

BMR is influenced by several factors: age, gender, height, body composition, fever / stress, nutritional state and ambient temperature.

Considerations

When assessing your body composition trend, it is necessary to keep the following in mind:

- A decrease in weight accompanied by no change in body fat or even an increase in body fat means that your body only lost water (for example, after exercising, time in the sauna or a limited diet for rapid weight loss).
- An increase in weight accompanied by a decrease or no change in body fat indicates the development of muscle mass.
- When the weight and body fat decrease at the same time it means that your diet is working and volume is being lost.

PRODUCT DESCRIPTION (see fig.1)

1. Display
2. Electrodes
3. "SET" button
4. "◀" "▶" buttons
5. Key to select unit of measurement (kg/lb/st)
6. Battery compartment

TECHNICAL SPECIFICATIONS

- Capacity: 180 kg / 396 lb / 28 st 4 lb
- Division: 100 g / 0.2 lb
- Personal data log areas for up to 10 users
- Two 3V lithium batteries CR2032
- LCD DISPLAY
- "Touch sensor" keys: touch activation
- Environmental conditions for use: +10°C +40°C; RH ≤85%
- Environmental storage conditions: +10°C +50°C; RH ≤85%

The scale calculates the body composition for individuals aged 10 to 80 measuring between 100 and 220 cm tall.

RECOMMENDATIONS FOR USE

- Always weigh yourself on the same scale at the same time, preferably when you wake up and while naked.
- It is important to weigh yourself with dry, bare feet positioned correctly on the electrodes (see figure). Also avoid contact between the legs so the scale operates correctly.
- Wait about 15 minutes after rising so water can distribute itself through your body.
- Scale precision could vary in the following cases:
 - a) high level of alcohol
 - b) high level of caffeine or drugs
 - c) after heavy meals
 - d) after rigorous sports activities
 - e) during illness
 - f) when pregnant
 - g) when dehydrated
 - h) with wet or unclean feet
 - i) swollen legs

INSTRUCTIONS FOR USE

FIRST USE

- This scale works with 2 replaceable 3V CR2032 lithium batteries.
- At first use, remove the batteries' protective film prior to use.

UNIT SELECTION

Turn on the scale and step on, checking the unit of measurement shown on the display (kg, lb or st). To change the unit of measurement, step off the scale and press the button located on the base of the scale.

USE

a. Measuring weight only

- 1) Place the scale on a flat, sturdy surface. An uneven or disjointed floor (e.g., rug, carpet, linoleum) may compromise the accuracy of the reading.
- 2) With the scale off, step on with parallel feet and your weight equally distributed (see images). Do not move until the scale has measured your weight. Do not lean on anything while the scale measures your weight in order to avoid altering the readings.
- 3) Step off the scale. The display will continue to show the reading for a few seconds before automatically turning off.

b. Entering personal data for calculating body composition

This scale can memorize data for 10 different people.

- 1) Activate the scale by pressing the centre of the platform. Press the "SET" key: the display shows the memory zone "P-01"/ "P-02", etc. which indicates person "01", "02", etc. To change the memory zone (memories from 1 to 10) press the keys "◀" or "▶" and confirm by pressing "SET".
- 2) Press the "◀" or "▶" buttons to enter the age (from 10 to 80), height (from 100 to 220 cm) and gender

(the display will show the outline of a man or woman) and confirm each entry by pressing "SET".

- 3) The scale will show the data entered in succession and then "0.0".

- To change the data you entered, recall the memory area and repeat the entire procedure.
- To enter data for another person, choose a free memory zone and repeat the entire procedure given above.
- During personal data entry operations, the scale will shut off automatically if no button is pressed within 15 seconds.

c. Body composition calculation

- 1) Activate the scale by pressing the centre of the scale. Select the desired memory zone by pressing key "◀" or "▶": the display shows "P-01" or "P-10". To change the memory zone (memories from 1 to 10), press the "◀" or "▶": the scale shows the data for the memory zone selected in success and then "0.0".
- 2) Step on the scale, making sure your bare feet are correctly positioned above the two electrodes. The scale shows the following in succession:
 - weight
 - body fat percentage (BF)
 - water percentage (BW)
 - muscle percentage (MM)
 - basal metabolism (Kcal)
- 3) The scale will shut off automatically after the second time the data above has been displayed.

Compare the body fat percentage (BF) with the parameters given in the "body fat table". If it is outside normal values, consult your family physician. Evaluate the water percentage (BW), as indicated in the "water %" paragraph, as well as muscle percentage (MM) as indicated in the "muscle %" paragraph.

Each time the scale is moved and then returned to its resting surface, the first weight must not be considered due to reorganization of the electronic board with the loading cells. Consider the second and successive weights, as in certain cases the first one could be altered for the reason mentioned above.

REPLACING BATTERIES

When the "LO" or the battery symbol appears on the display, replace the battery with a similar type.

1. Open the battery compartment, sliding the lid in the direction of the arrow.
2. Remove the worn battery, lifting it with the help of a tool if necessary. Dispose of the battery as indicated in the "Disposal procedure" paragraph.
3. Insert a new battery in accordance with the indicated polarity.
4. Close the battery compartment.

MAINTENANCE

- Store the appliance in a cool, dry area.
- Store the device in a horizontal position
- Only clean the scale with a soft, dry cloth.
- Never use chemicals or abrasives.

TROUBLESHOOTING

Problem	Possible cause	Solution
The scale doesn't turn on	You have not stepped onto the scale properly.	Step onto the scale with your feet in a parallel position. Attention: only turn on after stepping on the scale.
	The batteries are worn.	Replace the batteries.
"Lo" or the battery symbol is shown on the display	The batteries are low.	Replace them.
The display shows "Err".	Weight greater than the scale capacity.	Step off the scale immediately so as not to damage it and check the maximum capacity.
The display shows "Err2".	You did not step on the scale with bare feet.	Step onto the scale with bare feet.
	The skin on your feet is too dry.	Slightly dampen your feet.
	The personal data entered do not fall within the interval calculated by the device software.	Only use the scale to measure body weight.

All weights are approximate having been rounded up or down to the nearest whole number. We guarantee accuracy within +/-1% + 1 division.

DISPOSAL PROCEDURE

 The symbol on the bottom of the device indicates the separated collection of electric and electronic equipment (**Dir. 2012/19/Eu-WEEE**). At the end of life of the device, do not dispose it as mixed solid municipal waste, but dispose it referring to a specific collection centre located in your area or returning it to the distributor, when buying a new device of the same type to be used with the same functions. If the appliance to be disposed of is less than 25 cm, it can be returned to a retail location that is over 400 m² without having to purchase a new, similar device. This procedure of separated collection of electric and electronic devices is carried out forecasting a European environmental policy aiming at safeguarding, protecting and improving environment quality, as well as avoiding potential effects on human health due to the presence of hazardous substances in such equipment or to an improper use of the same or of parts of the same. Caution! The wrong disposal of electric and electronic equipment may involve sanctions.

To correctly dispose of batteries (**Dir. 2013/56/Eu, the Waste Batteries and Accumulators Regulations 2009**) **do not throw them into household waste** but rather dispose of them as special

waste at specialised recycling centres. For more information about disposal of used batteries, contact the store where you purchased the equipment containing the batteries, the municipality or the local waste disposal service.

 This product is meant for household use. The conformity, certified by the CE-marking on the device, refers to the Directive 2014/30/Eu on electromagnetic compatibility and on Directive 2011/65/Eu on the restriction of the use of certain hazardous substances in electrical and electronic equipment. This product is an electronic device that was tested to assure, according to the current technical knowledge, that it does not interfere with other nearby located equipment (electromagnetic compatibility) and that it is safe if used according to the advices given in the instructions for use. In case of operation anomalies, do not keep using the device and, if necessary, refer to the manufacturer directly.

 This product is meant for household use. The conformity, certified by the UKCA-marking on the device, refers to the Electromagnetic Compatibility Regulations 2016 and The Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations 2012. This product is an electronic device that was tested to assure, according to the current technical knowledge, that it does not interfere with other nearby located equipment (electromagnetic compatibility) and that it is safe if used according to the advices given in the instructions for use. In case of operation anomalies, do not keep using the device and, if necessary, refer to the manufacturer directly.

WARRANTY

This equipment is covered by a 2-year warranty starting from the purchase date, which has to be proven by means **of the stamp or signature of the dealer and by the fiscal payment receipt that shall be kept enclosed to this document**. This warranty period complies with the laws in force and applies only in case the consumer is an individual. Laica products are designed for domestic use only; therefore, their use for commercial purposes is forbidden. The warranty covers only production defects and does not apply, if the damage is caused by accidental events, misuse, negligence or improper use of the product. Use the supplied accessories only; the use of other accessories invalidate the warranty cover. Do not open the equipment for any reason whatsoever; the opening or the tampering of the equipment, definitely invalidate the warranty. Warranty does not apply to parts subject to tear and wear along the use, as well as to batteries, when they are included. Warranty decays after 2 years from the purchase date. In this case, technical service interventions will be carried out against payment. Information on technical service interventions, under warranty terms or against payment, may be requested by writing at info@laica.com. Repair or replacement of products covered by the warranty terms are free of charge. In case of faults, refer to the dealer; DO NOT send anything directly to LAICA. All interventions covered by warranty (including product replacement or the replacement of any of its parts) shall not be meant to extend the duration of the original warranty period of the replaced product. The manufacturer cannot be made liable for any damages that may be caused, directly or indirectly, to people, things and pets resulting from the inobservance of the provisions contained in the suitable user manual, especially with reference to the instructions related to installation, use and maintenance of the equipment. Being constantly involved in improving its own products, Laica reserves the right to modify its product, totally or partially, with reference to any production need, without that this involves any responsibility whatsoever by Laica or its dealers.

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