LAICA®

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PS7025

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LC168 - 10/2020

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Laica S.p.A. is under licence.
Other trademarks and trade name are those of their respective owners.

iOS, iPhone, iPad mini, iPod touch, App Store are trademarks of Apple Inc., registered in the U.S. and other countries - Android™, Play Store™ and Google Play™ are trademarks of Google Inc.

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SMART ELECTRONIC BATHROOM SCALE WITH CALCULATION OF % FAT, % WATER, % SKELETAL MUSCLE, BASAL METABOLIC RATE, BODY MASS INDEX, BODY BALANCE AND HEART RATE - INSTRUCTIONS AND WARRANTY

Dear customer, Laica would like to thank you for choosing one of our products, designed according to strict performance and quality criteria in order to ensure total satisfaction.

IMPORTANT READ CAREFULLY BEFORE USE KEEP IN A SAFE PLACE FOR FUTURE REFERENCE

The instruction manual must be considered an integral part of the product and must be kept for its entire life. If the product is transferred to another owner, its documentation must also be transferred in its entirety.

To ensure the safe and correct use of the product, the user must carefully read the instructions and warnings contained in the manual insofar as they provide important information regarding safety, user and maintenance instructions. Should the instruction manual get mislaid or you require additional information or clarifications, please fill in the relevant form on the website: <https://www.laica.it/> in the FAQ and Support section.

Electronic personal scale which measures your weight and calculate your body composition (percentage of fat, water, skeletal muscle and body mass index) and basal metabolic rate based on your height, age and gender. The personal scale also detects your body balance and heart rate. Compatible with bluetooth 4.0 technology or above, it allows you to automatically send the data to your smartphone so you can monitor your progress at any time. Technology featuring ITO (indium tin oxide) electrodes: innovative sensitive conductor coating which rapidly and accurately detects information about body composition.

INDEX

SYMBOLS KEY page 9
SAFETY WARNINGS page 9
BODY COMPOSITION page 10
PRODUCT DESCRIPTION page 11
INSTRUCTIONS FOR USE page 11
MAINTENANCE page 12
TROUBLESHOOTING page 12
DISPOSAL PROCEDURE page 13
WARRANTY page 13

SYMBOLS KEY

Warning

Prohibition

SAFETY WARNINGS

- Before using, make sure the device is intact with no visible damage. If you have any doubts, do not use the device and contact the retailer.
- Keep the plastic bag used for packaging out of reach of children: suffocation hazard.
- This device must only be used for its intended purpose and in the manner indicated in the instructions. All other uses are considered improper and therefore hazardous. The manufacturer cannot be considered responsible for any damage due to improper or incorrect use.
- This device can be used by children on their own from the age of 8 upwards.
- Individuals with reduced physical, sensory or mental capacities or unskilled people must only use the product if previously instructed on its use, in safety and only if informed of the dangers connected to the product itself. Children must not play with the device. Cleaning and maintenance operations must not be performed by children without adult supervision.
- In the event of a failure and/or malfunction, do not tamper with the device. Always contact your retailer for repairs.
- Handle the scale with care, protect it against knocks, extreme temperature fluctuations, humidity, dust, direct sunlight and sources of heat.
- ⊘ Do NOT allow liquids to penetrate into the body of the scale. This appliance is not waterproof. Do NOT leave and do NOT use the scale in water-saturated environments; humidity rates greater than 85% or contact with water or other liquids may cause corrosion or compromise proper scale operation.
- ⊘ Do NOT step onto the scale with wet feet, or if the surface of the scale is wet: slipping hazard.

ATTENTION! BEFORE USING THIS APPLIANCE

- **The percentage of body fat, water and skeletal muscle, the basal metabolic rate, data related to the body balance and heart rate calculated by the scale must be considered as indicative data only. For further information, consult a doctor or dietician.**
- **Self-measurement means checking, not diagnosis or treatment. Unusual readings must always be discussed with your physician. Under no circumstances should you alter the dosages of any medication prescribed by your physician.**
- **Through the electrodes located on the platform, the scale sends a weak, low intensity electrical current that passes through the body. The scale is not dangerous in any way.**
- **The scale is not recommended for the following individuals as the data detected by the scale may not be accurate:**
 - o individuals wearing a pacemaker or other electronic medical device
 - o people with fever
 - o children under 12 years of age
 - o pregnant women
 - o individuals undergoing dialysis, with oedema in the legs or osteoporosis
 - o people who take cardiovascular medication

SAFE USE OF BATTERIES

- Remove the batteries if the scale is not used for long periods of time and keep them in a well-ventilated and dry place, at room temperature.
- Do NOT recharge the batteries if they are not rechargeable.
- Do NOT recharge rechargeable batteries using methods other than those indicated in the manual or using unsuitable equipment.
- NEVER expose the batteries to sources of heat or direct sunlight.
- Failure to observe these instructions may result in damage to and/or the explosion of the batteries.
- Do NOT throw the batteries into an open flame.
- The removal or replacement of batteries must always be performed by an adult.
- Keep the batteries far from the reach of children: swallowing batteries carries the risk of death. If batteries are swallowed, immediately contact a doctor.
- The acid contained in the batteries is corrosive. Avoid contact with the skin, eyes or garments.

BODY COMPOSITION

The body composition is made up of:

- lean mass: all components without fat (water, muscle, connective tissue, and other non-fatty organic components), bone, bowels, organs, and blood;
- fatty mass: all fats that can be extracted from adipose tissue and other tissues/organs;
- bone mass.

Through the electrodes located on the platform, the scale sends a weak, low intensity electrical current that passes through the body. The different masses respond differently to the current: lean mass provides less resistance than the opposite. This body resistance is called bioelectrical impedance. The relative percentage of the various body masses varies based on gender, age, and physical stature.

Body fat % (info visible on the display of the scale and in the app)

The body fat percentage indicated in the table below represents the total fat that is calculated by adding essential fat and storage fat.

Essential fat is necessary for various bodily functions. The percentage is 3% in men and 12% in women. Storage fat gives energy, protects against physical trauma, and insulates the body to preserve heat. These percentages vary with the age and gender of the individual as shown in the table below. **If the percentages indicated by the scale do not fall within the values indicated as "normal" in the table, we recommend consulting with your family physician.**

BODYFAT TABLE					
	Age	UNDERWEIGHT	NORMAL	OVERWEIGHT	OBESE
WOMEN	10-39	14-19%	19.1-29%	29.1-34%	> 34.1%
	40-55	15-20%	20.1-30%	30.1-35%	> 35.1%
	56-80	16-21%	21.1-31%	31.1-36%	> 36.1%
MEN	10-39	11-19%	19.1-24%	24.1-29%	> 29.1%
	40-55	12-20%	20.1-25%	25.1-30%	> 30.1%
	56-80	13-21%	21.1-26%	26.1-31%	> 31.1%

% of muscle mass (info visible on the display of the scale and in the app)

There are three different types of muscle: smooth, striated skeletal, and striated cardiac.

Smooth muscle is made of smooth fibres.

These are found in the walls of the digestive tract, respiratory tract and genital tract and in the wall of the venous and arterial blood vessels.

Human striated skeletal muscle is in the skeletal and facial muscles. Cardiac striated muscle is found in the heart. This scale calculates the % of skeletal muscle and smooth muscle tissue in the body. On average, 40% of body mass is made up of skeletal muscle. Men usually have more pronounced muscles than women and the percentage may increase in those who practice sports.

The percentage of muscle starts to decrease with age.

Water % (info visible on the display of the scale and in the app)

The % of water in the body differs based on gender: in healthy adult women it ranges from 45% to 60% while in men it ranges from 50% to 65%.

The percentage of water is greater in childhood and decreases with age. Body weight variations can occur in a short period of time (hours), a medium period of time (days) or over a long period (weeks). Small variations in weight over a few hours is normal and due to water retention since extracellular water (which includes the interstitial fluid, plasma, lymph and transcellular fluid) is the only part capable of varying so rapidly. When weight losses occur in just a few days, it is very likely that it is part of the total water.

Basal metabolic rate (info visible in the app only)

The Basal Metabolic Rate, or BMR, is the minimum calories required for the human body to survive, in order to carry out basic functions such as circulation, breathing, digestion, and maintaining body temperature.

BMR is calculated on a person when at physical and mental rest, lying down on their back, having not eaten for 12 hours, after at least 8 hours' sleep and with a constant body temperature of 20°C.

Adult BMR varies from 1200 to 1800 kcal. BMR is influenced by many factors: age, gender, height, body composition, fever, stress, nutritional state and ambient temperature.

Body mass index (info visible on the display of the scale and in the app)

The body mass index (BMI) is the relation of body weight to height.

The classification given below is correlated to the risk that body weight exerts on health.

When BMI is between 18.5 and 24.9 it means that the weight does not statistically impact health and is to be considered normal. Any deviation, above or below this BMI interval should be

considered a health risk. In this case, we recommend consulting a physician. Getting in the habit of following a balanced diet definitely helps improve quality of life, promote health and prevent illness, increase vitality and maintain ideal weight!

It is important to know that BMI does not allow accounting for weight variations due to muscle mass, bone mass, water content or fat accumulation.

Therefore, a simple BMI calculation alone should not be misinterpreted, leading to "DIY" diets. The evaluation of the BMI should be made by a doctor or other specialists such as a nutritionist or dietitian.

Body balance (info only visible on the display of the scale)

The scale allows you to keep track of your personal body balance.

The display shows the percentage of weight respectively in the right and left part of the body and indicates the balance status with the following symbols:

SYMBOL	MEANING
≈	Body balance status
>	Possible body imbalance on left side
<	Possible body imbalance on right side

Considerations

When assessing your body composition trend, it is necessary to keep the following in mind:

- A decrease in weight accompanied by no change in body fat or even an increase in body fat means that your body only lost water (for example, after exercising, time in the sauna or a limited diet for rapid weight loss).
- An increase in weight accompanied by a decrease or no change in body fat indicates the development of muscle mass.
- When the weight and body fat decrease at the same time it means that your diet is working and volume is being lost.

PRODUCT DESCRIPTION (see fig.1)

- Display
 - Weight
 - Percentage of weight on left
 - Body balance symbol
 - Percentage of weight on right
 - % body fat
 - % water
 - Heart rate
 - Body mass index
 - % muscle mass
- Electrodes ITO
- Battery compartment

TECHNICAL CHARACTERISTICS

- Capacity: 180 kg / 400 lb
- Division: 100 g / 0.2 lb
- 4 disposable 1.5V AAA alkaline batteries
- LCD VA (Vertical Alignment) colour display
- Operating frequency: 2426-2460 MHz, effective radiated power: <0.9 dBm
- Environmental conditions for use: +10°C +40°C; RH ≤85%
- Environmental storage conditions: +10°C +50°C; RH ≤85%

	The scale calculates the body composition for individuals aged 12 to 99 measuring between 90 and 220 cm in height.
---	--

RECOMMENDATIONS FOR USE

- Always weigh yourself on the same scale at the same time, preferably when you wake up and before getting dressed.
- Wait about 15 minutes after getting up so the water can distribute itself through your body.
- It is important to weigh yourself with dry, bare feet positioned correctly on the electrodes (see figure). Avoid contact between the legs so the scale operates correctly.
- Scale precision could vary in the following cases:
 - high level of alcohol
 - high level of caffeine or drugs
 - after heavy meals
 - after rigorous sports activities
 - during illness
 - during pregnancy
 - when dehydrated
 - with wet or unclean feet
 - swollen legs

INSTRUCTIONS FOR USE**FIRST USE**

- This scale works with 4 disposable 1.5V AAA alkaline batteries.
- To insert the batteries, open the battery compartment by sliding the cover in the direction of the arrow.
- Insert the batteries according to the pole markings. Close the cover.

UNIT SELECTION

The unit of weight measurement can be selected in the "LAICA Home Wellness" app. The unit of measurement shown on the LCD display of the scale cannot be changed.

USE**a. Body weight measurement only**

- Place the scale on a flat, sturdy surface. An uneven or disjointed floor (e.g., rug, carpet, linoleum) may compromise the accuracy of the reading.
- With the scale turned off, step onto it with your feet parallel and your weight evenly distributed (see diagrams). Wait without moving until the scale has calculated your weight and displayed the body balance reading. Do not lean on anything while the scale measures your weight in order to avoid altering the readings.
- Step off the scale. The display will continue to show the reading for a few seconds before automatically turning off.

	To view the body composition data on the LCD display, the scale must be connected to the "LAICA Home Wellness" app.
---	---

b. Operation with app: calculation of body composition**List of compatible devices:**

- iOS: iPhone with 10 operating system.

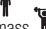
- **For device with Android™ operating system:** from version 6.0 (Marshmallow). Compatible with bluetooth 4.0 and higher technology, the scale sends data on body composition (fat percentage, water, skeletal muscle, basal metabolic data and body mass index) to the smartphone to better analyse personal weight composition.

When it connects to the app, the scale shows the body composition data on the app display as well. During the following operations, we recommend keeping the smartphone near the scale to keep the connection active.

1. Activate bluetooth in the smartphone settings
2. Install the free "LAICA Home Wellness" app on your smartphone
3. Start the app and follow the instructions to add the product and to create the new user by entering your personal data.
4. Turn on the scale by pressing the platform: the display will show "0.0".
5. Press the "DETECT WEIGHT" key on the app to connect the app to the scale and step onto the scale taking care to stand correctly on the ITO electrodes with bare feet. The scale shows:
 - a. your weight,
 - b. your body balance
6. Stand on the scale without moving until the LCD display shows the following data:

• your weight		
• your body balance		
• percentage of body fat		• percentage of water 
• body mass index		• percentage of muscle mass 
		• heart rate 

At the same time, the app shows:

- your weight
- target weight (if set in the app)
- percentage of body fat 
- percentage of water 
- basal metabolic rate 
- body mass index (BMI) 
- percentage of muscle mass 

	<ul style="list-style-type: none"> • If you step off the scale before all the data have been displayed, the app and the scale will not show the body composition data. • The scale displays the data with one decimal after the comma; the app displays the data with two decimals.
--	---

7. The scale switches off automatically after approximately 10 seconds of non-use. Compare the body fat percentage with the parameters given in the "body fat table". If it is outside normal values, consult your physician. Assess the percentage of water shown in the paragraph "% water" and the percentage of muscle mass shown in the paragraph "% muscle mass".

	<p>Starting from when you use the "Laica Home Wellness" scale for the second time, the scale will recognise the user from his/her weight (allowing for a weight variation of up to 2 kg). In this case, when the scale displays the body weight and balance, it automatically displays the data for the body composition, even without using the "LAICA Home Wellness" app. The data displayed by the scale without using the app are not recorded in the app and cannot be sent later.</p> <p>If there is more than one user with similar weights (less than 2 kg), the scale may use the parameters of another user. We recommend using the scale with the "LAICA Home Wellness" app to avoid confusing your body composition data with that of other users and ensure that it is recorded and monitored in the app.</p> <ul style="list-style-type: none"> • When you weigh yourself, make sure that the app is active and displayed on your smartphone so that you can check that the data has been promptly sent with your body composition. • Before taking a new reading, always make sure that the bluetooth connection on your smartphone is active. • For Android™ devices it is necessary to activate the localisation services. • Before weighing yourself for the first time, always select the user on your smartphone. • The body balance and heart rate are only displayed on the scale. • Each time the scale is moved or lifted and then placed back on its support surface, the first weighing must not be considered due to the fact that the electronic board needs to readjust with respect to the load cells. Consider the second and successive weights, as in certain cases the first one could be altered for the reason mentioned above.
---	--

BATTERY REPLACEMENT

- When the "LO" or battery symbol appears on the display, replace the batteries with the same type.
1. Open the battery compartment by sliding the cover in the direction of the arrow.
 2. Remove the worn batteries and dispose of them according to the instructions in the paragraph "Disposal Procedure".
 3. Insert the new batteries according to the pole markings.
 4. Close the battery compartment.

MAINTENANCE

- Store the device in a cool, dry area.
- Store the device in a horizontal position
- Only clean the scale with a soft, dry cloth.
- Never use chemicals or abrasive products

TROUBLESHOOTING

Problem	Possible cause	Solution
The scale doesn't turn on.	You have not stepped onto the scales properly.	Step onto the scale with your feet parallel. Note: the scale will turn on only after stepping onto it.
	The batteries have run out.	Replace the batteries.

Problem	Possible cause	Solution
"Lo" or the battery symbol is shown on the display.	The batteries are low.	Replace them.
The display shows "Err".	Weight greater than the scale capacity.	Step off the scale immediately so as not to damage it and check the maximum capacity.
The display only shows your weight and body balance.	You have weighed yourself for the first time without using the "LAICA Home Wellness" app.	To display all the body composition data, you must connect the scale to the "LAICA Home Wellness" app. Read the paragraph on "Operation with app: calculation of body composition."
	You stepped off the scale before the body composition data was displayed.	Stand on the scale without moving until all the data is shown on the display.
	You did not step on the scale with bare feet.	Step onto the scale with bare feet.
The skin on your feet is too dry.	The skin on your feet is too dry.	Slightly dampen your feet.
	The "LAICA Home Wellness" app doesn't display any data.	The bluetooth function has not been enabled in the smartphone settings.
The "LAICA Home Wellness" app only shows your body weight.	The smartphone went into standby during weighing.	Keep the app active on your smartphone during weighing to check that data is sent instantly.
	The "DETECT WEIGHT" button has not been pressed in the app.	Press the "DETECT WEIGHT" button in the app to send the data from the scale to the app.
The "LAICA Home Wellness" app only shows your body weight.	You did not step on the scale with bare feet.	Step onto the scale with bare feet.
	The skin on your feet is too dry.	Slightly dampen your feet.

All weights are approximate having been rounded up or down to the nearest whole number. We guarantee accuracy within +/-1% + 1 division.

DISPOSAL PROCEDURE

 The symbol on the bottom of the device indicates the separated collection of electric and electronic equipment (**Dir. 2012/19/Eu-WEEE**). At the end of life of the device, do not dispose it as mixed solid municipal waste, but dispose it referring to a specific collection centre located in your area or returning it to the distributor, when buying a new device of the same type to be used with the same functions. If the appliance to be disposed of is less than 25 cm, it can be returned to a retail location that is over 400 m² without having to purchase a new, similar device. This procedure of separated collection of electric and electronic devices is carried out forecasting a European environmental policy aiming at

safeguarding, protecting and improving environment quality, as well as avoiding potential effects on human health due to the presence of hazardous substances in such equipment or to an improper use of the same or of parts of the same. Caution! The wrong disposal of electric and electronic equipment may involve sanctions.

To correctly dispose of batteries (**Dir. 2013/56/Eu**) **do not throw them into household waste** but rather dispose of them as special waste at specialised recycling centres.

For more information about disposal of used batteries, contact the store where you purchased the equipment containing the batteries, the municipality or the local waste disposal service.

 This product is meant for domestic use. The conformity, certified by the CE-marking on the device, refers to the Directive 2014/30/Eu on electromagnetic compatibility. This product is an electronic device that was tested to assure, according to the current technical knowledge, that it does not interfere with other nearby located equipment (electromagnetic compatibility) and that it is safe if used according to the advices given in the instructions for use. In case of operation anomalies, do not keep using the device and, if necessary, refer to the manufacturer directly.

The manufacturer, Laica S.p.A., declares that the device is compliant with Directive 2014/53/Eu. The complete statement of compliance is available at the following Internet site: www.laica.it.

WARRANTY

This equipment is covered by a 2-year warranty starting from the purchase date, which has to be proven by means **of the stamp or signature of the dealer and by the fiscal payment receipt that shall be kept enclosed to this document.**

This warranty period complies with the laws in force and applies only in case the consumer is an individual. Laica products are designed for domestic use only; therefore, their use for commercial purposes is forbidden. The warranty covers only production defects and does not apply, if the damage is caused by accidental events, misuse, negligence or improper use of the product. Use the supplied accessories only; the use of other accessories invalidate the warranty cover. Do not open the equipment for any reason whatsoever; the opening or the tampering of the equipment, definitely invalidate the warranty. Warranty does not apply to parts subject to tear and wear along the use, as well as to batteries, when they are included. Warranty decays after 2 years from the purchase date.

In this case, technical service interventions will be carried out against payment. Information on technical service interventions, under warranty terms or against payment, may be requested by writing at info@laica.com. Repair or replacement of products covered by the warranty terms are free of charge. In case of faults, refer to the dealer; DO NOT send anything directly to LAICA.

All interventions covered by warranty (including product replacement or the replacement of any of its parts) shall not be meant to extend the duration of the original warranty period of the replaced product. The manufacturer cannot be made liable for any damages that may be caused, directly or indirectly, to people, things and pets resulting from the inobservance of the provisions contained in the suitable user manual, especially with reference to the instructions related to installation, use and maintenance of the equipment. Being constantly involved in improving its own products, Laica reserves the right to modify its product, totally or partially, with reference to any production need, without that this involves any responsibility whatsoever by Laica or its dealers.

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