HC96/c - 03/2023

LAICA S.p.A.
Viale del Lavoro, 10
36048 Barbarano Mossano (VI) - Italy
Tel. +39 0444.795314
info@laica.com
Made in China

www.laica.it

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NE2014

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PISTON NEBULIZER FOR AEROSOL THERAPY
Instructions and warranty

Dear Customer, Laica would like to thank you for the preference accorded to this product, designed according to the criteria of reliability and quality in order to meet your complete satisfaction.

IMPORTANT
PLEASE READ CAREFULLY BEFORE USE
KEEP FOR FUTURE REFERENCE

The instruction manual must be considered an integral part of the product and must be kept for its entire life. If the product is transferred to another owner, its documentation must also be transferred in its entirety.

To ensure the safe and correct use of the product, the user must carefully read the instructions and warnings contained in the manual insofar as they provide important information regarding safety, user and maintenance instructions. Should the instruction manual get mislaid or you require additional information or clarifications, please fill in the relevant form on the website: <https://www.laica.it/> in the FAQ and Support section.

Intended use and scope of application: This compressor device for aerosol therapy is an effective tool for the domestic treatment of upper and lower respiratory tract diseases. The device can be used on adult and paediatric patients. It is compact, easy to use and built in compliance with current European legislation concerning construction criteria for the safety of electromedical devices.

KEY TO SYMBOLS

Warning

Prohibition

Caution!
Read the user instructions carefully

Symbol for "Type BF applied parts"

0425

In compliance with European legislation on medical devices

Manufacturer

Production date

SN

Serial number

LOT

Batch production number

IP21:

Protection rating for enclosures of electrical equipment, whereby the first digit indicates the protection rating against the ingress of solid foreign objects (from 0 to 6) and the second digit indicates the protection rating against the infiltration of liquids (from 0 to 8).

SAFETY WARNINGS

- Prior to use, ensure the product is intact and that there is no visible damage. If in doubt, contact the retailer.
- Keep the plastic bag away from children: danger of suffocation.
- This device must be used only for its intended purpose and according to the user instructions. All other types of use shall be considered inappropriate and therefore hazardous. The manufacturer shall not be deemed liable for any damage caused by inappropriate or incorrect use.
- The use and maintenance of this product is only allowed by persons aged 14 years or over (capable of reading and understanding the instructions for use), and only if properly supervised by an adult in the case of persons with reduced physical, sensory or mental capacities, or unskilled persons. Children must not play with the device.
- **Keep the device away from children because it contains small parts that could be swallowed.**
- This device must be used on medical prescription.
- Before connecting the device, make sure the mains voltage corresponds to the voltage on the plate affixed to the product.
- ⚠ DO NOT use the device in the presence of a flammable anaesthetic mixture with air, oxygen, or nitrous oxide, harmful vapours, volatile substances or flammable gases.
- ⚠ NEVER leave the device operating unsupervised. Once finished using, turn it off and unplug it from the mains.
- Handle the product with care, protect it against accidental impact, extreme temperature fluctuations, humidity, dust, direct sunlight and sources of heat.
- In the event of a failure and/or malfunction, turn off the device without tampering with it. Always contact the retailer for repairs.
- Be sure to have dry hands before connecting or disconnecting the device and when pressing the "O/I" switch.

- ⊗ DO NOT pull the power cord to unplug the plug from the power outlet.
- ⊗ DO NOT use extensions, unwind the power cord and keep it away from sources of heat.
- ⊗ DO NOT tamper with the power cord for any reason. If damaged, contact your retailer.
 - Always use the "O/I" switch to turn the appliance off and then unplug. Disconnect the cord from the socket after use and before cleaning.
- ⊗ DO NOT immerse the device and DO NOT pour water or other liquids onto it.
 - If the device falls in the water, do not try to pick it up. Immediately unplug it. DO NOT reuse the device after removing it from the water.
- ⊗ DO NOT dry the device or its accessories in the microwave.
- ⊗ DO NOT use or leave the device in damp places or at temperatures above 40°C.
 - If the device is stored for a long period of time in a place with very low temperatures, wait at least two hours before turning it on.
 - Only use original accessories of the manufacturer.
 - Before each use, make sure that the parts have been assembled correctly.
- ⊗ DO NOT use the device if the air tube is bent.
- Make sure that the air filter is inserted correctly and that it is clean.
- ⊗ Do not lie down during inhalation. Do not tilt the nebulising bulb more than 45°.
- ⊗ Do not nebulise water.
- During use, do not touch the device for long periods, considering the potential damage due to overheating.
- ⊗ During use, DO NOT cover the device with towels or blankets. DO NOT insert any objects into the air intakes of the device.
- ⊗ To avoid damaging the nasal mucosa, DO NOT push the nosepiece into the rear part of the nose.
- Device not suitable for use with Pentamidine.

PRODUCT DESCRIPTION (see fig.1)

1. Main unit
2. On/off "O/I" switch
3. Compressor air connection
4. Air hose
5. Bulb hook
6. Child mask
7. Adult mask
8. Nosepiece
9. Mouthpiece
10. Nebulising bulb
 - a. Bulb top
 - b. Bulb cover to regulate the nebulisation flow rate

- c. Nebulisation nozzle
 - d. Bulb bottom
11. Air filter compartment lid
 12. Air filter

TECHNICAL SPECIFICATIONS

- Product name: Electrical piston compressor with thermal protector
- Trade name: NE2014
- Power supply: AC 230V ~ 50Hz, 150VA
- Operating pressure: 200Kpa-300Kpa (2.9psi-43.5psi)
- Compressor flow: 4-7 l/min.
- Noise: ≤65 db at a distance of one metre
- Nebulisation flow rate: ≥0.25 ml/min.
- MMAD: ≤ 5 micron
- Temporary use: 20 min. ON / 40 min. OFF.
- Life expectancy of the product in use (used to nebulise the medicine twice a day for about 10 minutes): about 500 hours. The frequent use of the device may shorten the life expectancy of the product.

Conditions during operation:

- Temperature: min. 10°C - max. 40°C, with air humidity from 30% to 85% and atmospheric pressure from 700 to 1060hPa

Storage conditions:

- Temperature: min. -20°C - max. 50°C, with relative air humidity from 30% to 85% and atmospheric pressure from 700 to 1060hPa

Bulb capacity:

- Min. filling volume: 2 ml
- Max. filling volume: 6 ml

INSTRUCTIONS FOR USE

- 1) Before using the device for the first time, clean it as described in the "Maintenance and disinfection" section.
- 2) To insert the medicine in the bulb, squeeze the bottom of the nebulising bulb and turn the top counter-clockwise to release it.
- 3) Remove the top, make sure that the nebulisation nozzle (no.10c of the exploded diagram) is correctly attached to the bulb top (see Fig.2) and insert the amount of medicine prescribed by the doctor into the bottom of the nebulising bulb.
- 4) After inserting the medicine, screw the top back onto the bottom. Put the bulb on a clean and dry surface.
- 5) Insert the plug in the socket.

- 6) Firmly connect one end of the air tube to the bulb and insert the other end into the main unit tube connection.
- 7) Apply the selected accessory: mask, mouthpiece or nosepiece as prescribed by the doctor.
- 8) Turn on the device by pressing the "O/I" switch. Place the mouthpiece in the mouth or the mask on the face, taking care to cover the mouth and nose, or place the nosepiece in the nostrils.
- 9) Inhale and exhale normally during the treatment. Sit upright in a relaxed position. **Do not lie down during inhalation. Do not tilt the nebulising bulb more than 45°.** Stop the inhalation if you feel sick.
- 10) The speed of nebulisation of the medicine can be adjusted by turning the bulb cover: turn it clockwise to widen the opening of the bulb and increase the flow rate, turn it counter-clockwise to close the opening and decrease the flow rate. **With the nosepiece accessory, use the minimum nebulisation speed (bulb cover closed).**
- 11) At the end of inhalation, turn the device off by pressing the "O/I" key and disconnect the plug from the power outlet
- 12) Remove the air tube from the bulb and from the main unit tube connection.

IMPORTANT: Like most aerosol therapy devices, at the end of the inhalation treatment, a certain amount of medicine will remain in the bulb. This is perfectly normal. This amount of the medicine, also called the residual volume, cannot be atomised.
- 13) Empty out any residual medicine from the bulb, clean all parts used as described in the paragraph "Maintenance and disinfection".

- ⚠ **This device was designed for intermittent operation - 20 min On / 40 min Off.** Turn the device off after 20 minutes of operation and wait another 40 minutes before performing another treatment.
- During use, the main unit may heat up; this is perfectly normal.

MAINTENANCE AND DISINFECTION

- Wash your hands thoroughly before cleaning and disinfecting the accessories.
- Carefully clean all of the parts and remove the medicine residues and possible impurities after every treatment.
- Clean the accessories if they have not been used for a long time.
- ⊗ The main unit and the air tube must be cleaned externally using a clean, damp cloth. Never clean the device under running water or by immersion. Device with housing not protected against the penetration of liquids.
- **Cleaning with water:** Clean all accessories in hot water, except for the air tube, and leave them to dry naturally.
- **Disinfection:** All of the accessories, except for the air hose, can be disinfected with chemical

- disinfectants according to the doses and limitations indicated by the disinfectant manufacturer. Submerge all the accessories, except for the air hose, in the disinfectant solution for a specified period (we recommend a contact time of less than 10 seconds). Completely remove the residues of the disinfectant by thoroughly rinsing the accessories under running drinking water and allow them to air dry.
- DO NOT BOIL THE ACCESSORIES IN WATER.** DO NOT sterilise with autoclaves, Ethylene Oxide (EO) gas, low-temperature plasma sterilisers or steam sterilisers.
- **Cleaning the air tube:** clean the external part using a clean damp cloth. To eliminate any moisture in the air tube, connect one of the two ends to the main unit and turn the device on for a few seconds without connecting the tube to the bulb.
 - **Replacing the air filter:** Replace the air filter when it becomes dark in colour and when it has been used for more than 60 treatments. To replace the filter, lift the air filter compartment cover, remove the used filter and insert the new one. Before inserting a new air filter, always make sure that the air filter compartment cover is clean. It is NOT possible to wash used air filters. DO NOT insert cotton instead of the filters. DO NOT use the product without a filter.
 - **Microbial contamination:** In the presence of illnesses with a risk of infection and microbial contamination, we recommend personal use of the accessories and nebulising bulb (always consult your physician).
- ⊗ Do not bend the air tube.
 - Store the device in a cool, dry area.

TROUBLESHOOTING		
Problem	Possible cause	Solution
The device does not turn on	The device is not plugged in	Plug the device into a power outlet
The device is on but does not nebulise or does not nebulise enough.	There is no medicine in the bulb or the amount of medicine is lower or higher than the capacity of the bulb.	Insert the medicine into the bulb, bearing in mind its capacity (between 2 and 6 ml)
	The nebulisation nozzle was not inserted in the bulb (no. 10c)	Insert the nebulisation nozzle in the bulb as described in fig. 2
	The nebulisation nozzle is clogged	Rinse the nebulisation nozzle under hot running water. DO NOT insert sharp objects such as needles into the nebulisation nozzle

Problem	Possible cause	Solution
The device is on but does not nebulise or does not nebulise enough.	The nebulising bulb has not been assembled properly	Assemble the nebulising bulb as described in fig. 3
	The nebulising bulb has been tilted too much during use	DO NOT tilt the nebulising bulb more than 45°.
	The air tube is bent or crushed	Extend the air tube
	The air tube is not connected properly	Insert the two ends of the air tube properly
	The air filter is clogged or dirty	Replace the air filter
The device is noisier than usual	The air filter compartment cover is not inserted correctly	Close the air filter compartment cover correctly
The device overheats during use	The device is covered with towels or blankets	Do not cover the device during use
	The device has been used continuously for over 20 minutes	Turn the device off after 20 minutes of operation and allow it to cool for 40 minutes.

N.B. If the appliance does not resume its correct functioning despite the checks performed, contact your dealer.

DISPOSAL PROCEDURE (Dir. 2012/19/Eu-WEEE)

 The symbol on the bottom of the device indicates the separated collection of electric and electronic equipment.
At the end of life of the device, do not dispose it as mixed solid municipal waste, but dispose it referring to a specific collection centre located in your area or returning it to the distributor, when buying a new device of the same type to be used with the same functions. If the appliance to be disposed of is less than 25 cm, it can be returned to a retail location that is over 400 m² without having to purchase a new, similar device. This procedure of separated collection of electric and electronic devices is carried out forecasting a European environmental policy aiming at safeguarding, protecting and improving environment quality, as well as avoiding potential effects on human health due to the presence of hazardous substances in such equipment or to an improper use of the same or of parts of the same.
Caution! The wrong disposal of electric and electronic equipment may involve sanctions.

WARRANTY

This appliance is guaranteed for 2 years from the time of delivery of the goods, or another longer term envisaged by the national legislation of the consumer's residence. This provision complies with Italian and European legislation. The Laica products are designed for home use and must not be used in public venues. The warranty only covers manufacturing defects and does not apply if the damage is caused by an accidental event, incorrect use, negligence or misuse of the product. Use only the accessories supplied; the use of different accessories may result in invalidity of the warranty. Do not open the unit for any reason; in the case of opening or tampering, the warranty is definitively voided. This warranty does not apply to parts subject to wear or to the batteries when supplied. After 2 years from delivery, or another longer term envisaged by the national legislation of the consumer's residence, the warranty expires; in this case, the technical assistance interventions will be carried out against a fee. Information on technical assistance, whether under warranty or for a fee, can be requested by contacting info@laica.com.
No payment will be due for repairs or replacements of products that fall within the terms of the warranty. In the event of faults, contact the retailer. Do NOT send the appliance directly to LAICA. All the operations under warranty (including those of replacement of the product or part thereof) will not prolong the duration of the original period of warranty of the product replaced. The manufacturer declines any liability for any damage that may, directly or indirectly, be caused to persons, property or animals as a result of the non-observance of all the requirements established in the relevant instructions manual and concerning, especially, warnings relating to installation, use and maintenance of the appliance. Laica, in its constant commitment to improving its products, is entitled to changing without any notice, in whole or in part, its products in relation to production requirements, without this entailing any liability for Laica towards its dealers. For further information: www.laica.it.

Guidelines and declaration of the manufacturer

1. This device must be installed and operated in accordance with the information supplied in the ACCOMPANYING DOCUMENTATION;
2. IMMUNITY TEST LEVELS for the basic safety and essential performance of ME equipment and ME systems must be selected with a view toward the high probability of maintaining basic safety and essential performance, and must be consistent with the environment of the professional healthcare structure, domestic healthcare environment and special environments - based on the intended place of use.
3. DOMESTIC HEALTHCARE ENVIRONMENT implies a residential space in which the patient normally resides, or other spaces where patients are present and which are not environments of professional healthcare structures, where healthcare operators with medical training are continuously available when there are patients. This may be for example schools, external environments, places of domicile, vehicles, hotels and pensions.
EXAMPLE: As indicated in table 6 of the standard IEC 60601-1-2:2014 for ME DEVICES, a common cell phone with a maximum output power of 2 W has a yield of d = 3.3 m at IMMUNITY level RI 3 V/m.

A1 Electromagnetic Emissions - For all EQUIPMENT and SYSTEMS

Guidance and manufacture's declaration electromagnetic emission		
The DEVICE is intended for use in the electromagnetic environment specified below. The customer or the user of the DEVICE should assure that it is used in such an environment.		
Emission test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The DEVICE uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emission CISPR 11	Class B	
Harmonic emissions IEC 61000-3-2	Class A	The DEVICE is suited for use in any structure constituted by domestic environments or environments directly connected to the public low-voltage power mains used to supply domestic buildings.
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

A2 Electromagnetic Immunity - For Home Healthcare Environment EQUIPMENT and SYSTEMS.

Guidance and manufacture's declaration electromagnetic immunity			
The DEVICE is intended for use in the electromagnetic environment specified below. The customer or the user of DEVICE should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air	±8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air	Floors should be wood, concrete or ceramic tile. If floor are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines	±2 kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV line to line ±2 kV line to ground	±1 kV line to line ±2 kV line to ground	
Power frequency (50/60Hz) magnetic field IEC 61000-4-8	30A/m	30A/m	
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0 % UT; 0,5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°	0 % UT; 0,5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°	Mains power quality should be that of a typical commercial or hospital environment. If the user of the DEVICE requires continued operation during power mains interruptions, it is recommended that the DEVICE be powered from an uninterruptible power supply.
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms	The MANUFACTURER should consider reducing the minimum separation distance, based on RISK MANAGEMENT, and using higher IMMUNITY TEST LEVELS that are appropriate for the reduced minimum separation distance. Minimum separation distances for higher IMMUNITY TEST LEVELS shall be calculated using the following equation: $E = \frac{6}{d} \sqrt{P}$ Where P is the maximum power in W, d is the minimum separation distance in m, and E is the IMMUNITY TEST LEVEL in V/m.
Radiated RF IEC 61000-4-3	10 V/m 80 MHz to 2.7 GHz	10 V/m	
NOTE UT is the a.c. mains voltage prior to application of the test level			

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