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MECHANICAL PERSONAL SCALES

INSTRUCTIONS AND WARRANTY

IMPORTANT
READ CAREFULLY BEFORE USE
KEEP IN A SAFE PLACE FOR FUTURE REFERENCE

The instruction manual must be considered an integral part of the product and must be kept for the entire life. If the product is transferred to another owner, its documentation must also be transferred in its entirety.
To ensure the safe and correct use of the product, the user must carefully read the instructions and warnings contained in the manual insofar as they provide important information regarding safety, user and maintenance instructions.
Should the instruction manual get mislaid or you require additional information or clarifications, please fill in the relevant form on the website: <https://www.laica.it/> in the FAQ and Support section.

SYMBOLS KEY	
	Warning
	Prohibition

SAFETY WARNINGS

- Before using, make sure the device is intact with no visible damage. If you have any doubts, do not use the device and contact the retailer.
- Keep the plastic bag used for packaging out of reach of children: suffocation hazard.
- This device must only be used for its intended purpose and in the manner indicated in the instructions. All other uses are considered improper and therefore hazardous. The manufacturer cannot be considered responsible for any damage due to incorrect or improper use.
- This device can be used by children on their own from the age of 8 upwards.
- This device can be used by persons with limited physical, sensory or mental capabilities, or by inexperienced people, only if they have been previously instructed about its safe use and only if they have been informed about the dangers of the product itself. Do not allow children to play with the device.
- Cleaning and maintenance operations cannot be carried out by children without supervision from an adult.
- In the event of faults and/or malfunctions, do not tamper with the machine. For repairs, always contact your reseller.
- Handle the scale with care, protect it against knocks, extreme temperature fluctuations, humidity, dust, direct sunlight and sources of heat.
- Do NOT allow liquids to penetrate into the body of the scale. This appliance is not waterproof. Do NOT leave and do NOT use the scale in water-saturated environments: humidity rates greater than 85% or contact with water or other liquids may cause corrosion or compromise proper scale operation.
- Do NOT step onto the scale with wet feet, or if the surface of the scale is wet: slipping hazard.

PRODUCT DESCRIPTION (see fig.1)	
1. Platform	
2. Dial	
3. Scale needle	
4. Zero wheel	

TECHNICAL SPECIFICATIONS	
Tolerance: +/-1 division per weighing lower than or equal to 50 kg, +/-2 divisions per weighing above 50 kg	
Environmental operating conditions: +10°C +40°C; RH 85%	
Environmental storage conditions: +10°C +50°C; RH 85%	

RECOMMENDATIONS FOR USE

Always weigh yourself at the same time of day, preferably when waking up and undressed.

INSTRUCTIONS FOR USE	
USE	
1) Position the scales on a flat and solid surface. The presence of an uneven or soft floor (e.g. carpet, rug, lin) may affect the accuracy of the measurement.	
2) Before stepping on to the scales, exert pressure on them 2 or 3 times and ensure that the needle indicates 0, otherwise zero with the zero wheel.	
3) Step on to the scales with your feet side by side and weight evenly distributed (see diagram). Wait without moving until the scales calculate your weight. Do not rest on anything during measurement to avoid affecting the readings taken.	
4) Step off the personal scales, the needle automatically returns to 0.	

MAINTENANCE	
Store the device in a cool and dry place.	
Store the device in a horizontal position.	
Clean the platform of the scales with a damp cloth and the bottom with a soft dry cloth.	
Never use chemical or abrasive products.	

TROUBLESHOOTING	
Problem	Possible cause
The needle of the scale doesn't go back to 0	Scales not calibrated
	Zero the scales with the zero wheel

WARRANTY	

This appliance is guaranteed for 2 years from the time of delivery of the goods, or another longer term envisaged by the national legislation of the consumer's residence. This provision complies with Italian and European legislation. The Laica products are designed for home use and must not be used in public venues. The warranty only covers manufacturing defects and does not apply if the damage is caused by an accidental event, incorrect use, negligence or misuse of the product. Use only the accessories supplied; the use of different accessories may result in invalidity of the warranty. Do not open the unit for any reason; in the case of opening or tampering, the warranty is definitively voided. This warranty does not apply to parts subject to wear or to the batteries when supplied. After 2 years from delivery, or another longer term envisaged by the national legislation of the consumer's residence, the warranty expires; in this case, the technical assistance interventions will be carried out against a fee. Information on technical assistance, whether for warranty or for a fee, can be requested by contacting info@laica.com.
No payment will be due for repairs or replacements of products that fall within the terms of the warranty. In the event of faults, contact the retailer. Do NOT send the appliance directly to LAICA. All the operations under warranty (including those of replacement of the product or part thereof) will not prolong the duration of the original period of warranty of the product replaced. The manufacturer declines any liability for any damage that may, directly or indirectly, be caused to persons, property or animals as a result of the non-observance of all the requirements established in the relevant instructions manual and concerning, especially, warnings relating to installation, use and maintenance of the appliance. LAICA, in its constant commitment to improving its products, is entitled to changing without any notice, in whole or in part, its products in relation to production requirements, without this entailing any liability for Laica towards its dealers. For further information: www.laica.it.

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