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TARE
If the baby is laid on a towel or cover, use the TARE function. Turn the scale on with the ON key. The display shows "8888". Wait until the display shows "0.00" then place the towel or cover on the plate. Press the TARE key and wait until the display shows "0.00"; the scale is now ready for use.

UTILIZING THE SCALE WITH THE PLATE
Remove the 4 green rubber plugs. Fix the plate to the scale using the supplied locking nut (see the illustration). Press the ON key. The display shows "8888". Wait until the display shows "0.00"; the scale is now ready for use. Lay the baby on the plate and wait until the display shows its weight. Press the OFF key to turn the scale off, otherwise the scale turn off automatically after 2 minutes without being used.

MAINTENANCE
• Take care of your scale, as it is a precision instrument. Therefore, avoid exposing it to extreme temperatures, damp, knocks, dust and direct sunlight.
• Do not drop or knock the scale.
• Keep the scale away from sources of heat.
• **Do not jump on the scale**
• Clean the platform with a damp cloth.
• Clean the bottom of the scale with just a soft dry cloth.
• Never use chemical or abrasive products.
• Take care that no liquids penetrate into the scale, as it is not waterproof. Do not leave or use the scale in very damp surroundings, humidity above 85% or contact with water and other liquids could cause corrosion and effect the precision of the scale.

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DISPOSAL PROCEDURE (Dir. 2012/19/ Ue-WEEE)
The symbol on the bottom of the device indicates the separated collection of electric and electronic equipment. At the end of life of the device, do not dispose it as mixed solid municipal waste, but dispose it referring to a specific collection centre located in your area or returning it to the distributor, when buying a new device of the same type to be used with the same functions. If the appliance to be disposed of is less than 25 cm, it can be returned to a retail location that is over 400 m² without having to purchase a new, similar device. This procedure of separated collection of electric and electronic devices is carried out forecasting a European environmental policy aiming at safeguarding, protecting and improving environment quality, as well as avoiding potential effects on human health due to the presence of hazardous substances in such equipment or to an improper use of the same or of parts of the same. Caution! The wrong disposal of electric and electronic equipment may involve sanctions.

WARRANTY
This equipment is covered by a 2-year warranty starting from the purchase date, which has to be proven by means of the **stamp or signature of the dealer and by this fiscal payment receipt that shall be kept enclosed in this document.**
This warranty period complies with the laws in force and applies only in case the consumer is an individual. Laica products are designed for domestic use only; therefore, their use for commercial purposes is forbidden. The warranty covers only production defects and does not apply, if the damage is caused by accidental events, misuse, negligence or improper use of the product. Use the supplied accessories only; the use of other accessories invalidate the warranty cover. The opening or the tampering of the equipment, definitely invalidate the warranty. Warranty does not apply to parts subject to tear and wear along the use, as well as to batteries, when they are included.
Warranty decays after 2 years from the purchase date. In this case, technical service interventions will be carried out against payment. Information on technical service interventions, under warranty terms or against payment, may be requested by writing at info@laica.com. Repair or replacement of products covered by the warranty terms are free of charge. In case of faults, refer to the dealer; DO NOT send anything

YOU ARE STRONGLY RECOMMENDED TO READ THE INSTRUCTIONS AND WARNINGS HEREIN BEFORE USING THE EQUIPMENT AND TO KEEP THE BOOKLET IN A SAFE PLACE.

TIPS FOR USE
Place the scale on a flat and hard surface.
A soft and uneven floor(e.g. carpets, linoleum) could cause an alteration to the weight reading.
If the scale is used without plate, the baby has to stand on the scale with the feet parallelly positioned and the weight must be equally distributed (see the drawing), before the zeros disappear.
Pay attention the baby stands still while the scale measures the weight. Pay attention that the baby does not lean on anything while being measured, as this could alter the reading.

EN English	ELECTRONIC BABY SCALE Instructions and warranty
<p>directly to LAICA. All interventions covered by warranty (including product replacement or the replacement of any of its parts) shall not be meant to extend the duration of the original warranty period of the replaced product. The manufacturer cannot be made liable for any damages that may be caused, directly or indirectly, to people, things and pets resulting from the inobservance of the provisions contained in the suitable user manual, especially with reference to the instructions related to installation, use and maintenance of the equipment. Being constantly involved in improving its own products, Laica reserves the right to modify its product, totally or partially, with reference to any production need, without that this involves any responsibility whatsoever by Laica or its dealers.</p>	

directly to LAICA.
All interventions covered by warranty (including product replacement or the replacement of any of its parts) shall not be meant to extend the duration of the original warranty period of the replaced product.
The manufacturer cannot be made liable for any damages that may be caused, directly or indirectly, to people, things and pets resulting from the inobservance of the provisions contained in the suitable user manual, especially with reference to the instructions related to installation, use and maintenance of the equipment.
Being constantly involved in improving its own products, Laica reserves the right to modify its product, totally or partially, with reference to any production need, without that this involves any responsibility whatsoever by Laica or its dealers.

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