LAICA®

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PS7020

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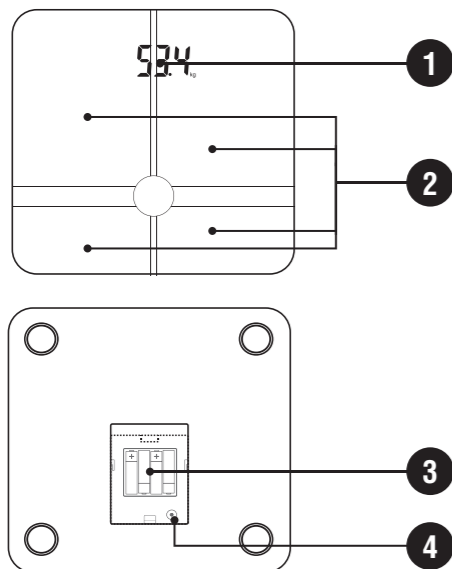
PS7020

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LC165 - 11/2019

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Laica S.p.A. is under licence.
Other trademarks and trade name are those of their respective owners.

iOS, iPhone, iPad mini, iPod touch, App Store are trademarks of Apple Inc., registered in the U.S. and other countries - Android™, Play Store™ and Google Play™ are trademarks of Google Inc.

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SMART ELECTRONIC SCALE FOR BODY FAT %, WATER %, MUSCLE %, BASAL METABOLIC RATE WITH BODY MASS INDICATOR – INSTRUCTIONS AND WARRANTY

Dear customer, Laica would like to thank you for choosing one of our products, designed according to strict performance and quality criteria in order to ensure total satisfaction.

IMPORTANT READ CAREFULLY BEFORE USE KEEP IN A SAFE PLACE FOR FUTURE REFERENCE

The instruction manual must be considered an integral part of the product and must be kept for its entire life. If the appliance is transferred to another owner, its documentation must also be transferred in its entirety.

To ensure the safe and correct use of the product, the user must carefully read the instructions and warnings contained in the manual insofar as they provide important information regarding safety, user and maintenance instructions. If the instruction manual is lost or in the event you require additional information or clarifications, please contact the company at the address shown below:

Laica S.p.A. - Viale del Lavoro, 10 - 36048 Barbarano Mossano (VI) – Italy - Tel. +39 0444.795314 - info@laica.com - www.laica.it

Electronic bathroom scale that measures the weight and calculates the body composition (percentage of body fat, water, muscle, and BMI) and the basal metabolic rate based on height, age and gender. Compatible with bluetooth 4.0 and higher technology, it allows you to automatically send the data to your smartphone so you can monitor your progress at any time. With ITO (indium tin oxide) electrodes: innovative sensitive conductor coating which rapidly and accurately detects information about body composition.

INDEX

SYMBOLS KEY	page 8
SAFETY WARNINGS	page 8
BODY COMPOSITION	page 9
PRODUCT DESCRIPTION	page 9
INSTRUCTIONS FOR USE	page 10
MAINTENANCE	page 10
TROUBLESHOOTING	page 11
DISPOSAL PROCEDURE	page 11
WARRANTY	page 11

SYMBOLS KEY

- Warning
- Prohibition

SAFETY WARNINGS

- Before using, make sure the device is intact with no visible damage. If you have any doubts, do not use the device and contact the retailer.

- Keep the plastic bag used for packaging out of reach of children: suffocation hazard.
- This device must only be used for its intended purpose and in the manner indicated in the instructions. All other uses are considered improper and therefore hazardous.
- The manufacturer cannot be considered responsible for any damage due to improper or incorrect use.
- This device can be used by children on their own from the age of 8 upwards.
- Individuals with reduced physical, sensory or mental capacities or unskilled people must only use the product if previously instructed on its use, in safety and only if informed of the dangers connected to the product itself.
- Children must not play with the device.
- Cleaning and maintenance operations must not be performed by children without adult supervision.
- In the event of a failure and/or malfunction, do not tamper with the device.
- Always contact your retailer for repairs.
- Handle the scale with care, protect it against knocks, extreme temperature fluctuations, humidity, dust, direct sunlight and sources of heat.
- ⊘ Do NOT allow liquids to penetrate into the body of the scale.
- This appliance is not waterproof. Do NOT leave and do NOT use the scale in water-saturated environments; humidity rates greater than 85% or contact with water or other liquids may cause corrosion or compromise proper scale operation.
- ⊘ Do NOT step onto the scale with wet feet, or if the surface of the scale is wet: slipping hazard.

	ATTENTION! BEFORE USING THIS APPLIANCE
--	---

- **The body fat, water, and skeletal muscle percentages and the basal metabolism calculated by the scale must only be considered to be indicative data. For further information, consult a doctor or dietician.**
- **Through the electrodes located on the platform, the scale sends a weak, low intensity electrical current that passes through the body. The scale is not dangerous in any way.**
- **The scale is not recommended for the following individuals as the data detected by the scale may not be accurate:**
 - ⊘ individuals wearing a pacemaker or other electronic medical device
 - ⊘ people with fever
 - ⊘ children under 12 years of age
 - ⊘ pregnant women
 - ⊘ individuals undergoing dialysis, with oedema in the legs or osteoporosis
 - ⊘ people who take cardiovascular medication

SAFE USE OF BATTERIES

- Remove the batteries if the scale is not used for long periods of time and keep them in a well-ventilated and dry place, at room temperature.
- ⊘ Do NOT recharge the batteries if they are not rechargeable.
- ⊘ Do NOT recharge rechargeable batteries using methods other than those indicated in the manual or using unsuitable equipment.
- ⊘ NEVER expose the batteries to sources of heat or direct sunlight.
- Failure to observe these instructions may result in damage to and/or the explosion of the batteries.
- ⊘ Do NOT throw the batteries into an open flame.
- The removal or replacement of batteries must always be performed by an adult.
- Keep the batteries far from the reach of children: swallowing batteries carries the risk of death. If batteries are swallowed, immediately contact a doctor.
- The acid contained in the batteries is corrosive. Avoid contact with the skin, eyes or garments.

BODY COMPOSITION

The body composition is made up of:

- lean mass: all components without fat (water, muscle, connective tissue, and other non-fatty organic components), bone, muscles, bowels, water, organs, and blood;
- fatty mass: all fats that can be extracted from adipose tissue and other tissues/organs;
- bone mass.

Through the electrodes located on the platform, the scale sends a weak, low intensity electrical current that passes through the body. The different masses respond differently to the current: lean mass provides less resistance than the opposite. This body resistance is called bioelectrical impedance. The relative percentage of the various body masses varies based on gender, age, and physical stature.

Body fat %

The body fat percentage indicated in the table below represents the total fat that is calculated by adding essential fat and storage fat. Essential fat is necessary for various bodily functions. The percentage is 3% in men and 12% in women. Storage fat gives energy, protects against physical trauma, and insulates the body to preserve heat. These percentages vary with the age and gender of the individual as shown in the table below. **If the percentages indicated by the scale do not fall within the values indicated as "normal" in the table, we recommend consulting with your family physician.**

BODYFAT TABLE					
	Age	UNDERWEIGHT	NORMAL	OVERWEIGHT	OBESE
WOMEN	10-39	14-19%	19.1-29%	29.1-34%	> 34.1%
	40-55	15-20%	20.1-30%	30.1-35%	> 35.1%
	56-80	16-21%	21.1-31%	31.1-36%	> 36.1%
MEN	10-39	11-19%	19.1-24%	24.1-29%	> 29.1%
	40-55	12-20%	20.1-25%	25.1-30%	> 30.1%
	56-80	13-21%	21.1-26%	26.1-31%	> 31.1%

% of muscle mass (% of skeletal muscle)

There are three different types of muscle: smooth, striated skeletal, and striated cardiac. Smooth muscle is made of smooth fibres. These are found in the walls of the digestive tract, respiratory tract and genital tract and in the wall of the venous and arterial blood vessels. Human striated skeletal muscle is in the skeletal and facial muscles. Cardiac striated muscle is found in the heart. This scale calculates the % of skeletal muscle and smooth muscle tissue in the body. On average, 40% of body mass is made up of skeletal muscle. Men usually have more pronounced muscles than women and the percentage may increase in those who practice sports. The percentage of muscle starts to decrease with age.

Water %

The % of water in the body differs based on gender: in healthy adult women it ranges from 45% to 60% while in men it ranges from 50% to 65%. The percentage of water is greater in childhood and decreases with age. Body weight variations can occur in a short period of time (hours), a medium period of time (days) or over a long period (weeks). Small variations in weight over a few hours is normal and due to water retention since extracellular water (which includes the interstitial fluid, plasma, lymph and transcellular fluid) is the only part capable of varying so rapidly. When weight losses occur in just a few days, it is very likely that it is part of the total water.

Basal metabolic rate

The Basal Metabolic Rate, or BMR, is the minimum calories required for the human body to survive, in order to carry out basic functions such as circulation, breathing, digestion, and maintaining body temperature. BMR is calculated on a person when at physical and mental rest, lying down on their back, having not eaten for 12 hours, after at least 8 hours' sleep and with a constant body temperature of 20°C. Adult BMR varies from 1200 to 1800 kcal. BMR is influenced by many factors: age, gender, height, body composition, fever / stress, nutritional state and ambient temperature.

Body mass index

BMI TABLE	BMI VALUE
UNDERWEIGHT	<18.5
NORMAL	18.5-24.9
OVERWEIGHT	25.0-29.9
OBESE	>30

The body mass index (BMI) is the relation of body weight to height. The classification given below is correlated to the risk that body weight exerts on health. When BMI is between 18.5 and 24.9 it means that the weight does not statistically impact health and is to be considered normal.

Any deviation, above or below this BMI interval should be considered a health risk. In this case, we recommend consulting a physician. Getting in the habit of following a balanced diet definitely helps improve quality of life, promote health and prevent illness, increase vitality and maintain ideal weight! It is important to know that BMI does not allow accounting for weight variations due to muscle mass, bone mass, water content or fat accumulation.

Therefore, a simple BMI calculation alone should not be misinterpreted, leading to "DIY" diets. The evaluation of the BMI should be made by a doctor or other specialists such as a nutritionist or dietician.

Considerations

- When assessing your body composition trend, it is necessary to keep the following in mind:
- A decrease in weight accompanied by no change in body fat or even an increase in body fat means that your body only lost water (for example, after exercising, time in the sauna or a limited diet for rapid weight loss).
 - An increase in weight accompanied by a decrease or no change in body fat indicates the development of muscle mass.
 - When the weight and body fat decrease at the same time it means that your diet is working and volume is being lost.

PRODUCT DESCRIPTION (see fig.1)

1. Display
2. ITO electrodes
3. Battery compartment
4. Unit of measurement selection key

TECHNICAL SPECIFICATIONS

- Capacity: 180 kg / 400 lb
- Division: 100 g / 0.2 lb
- 4 disposable 1.5V AAA alkaline batteries
- LED DISPLAY
- Operating frequency: 2402-2480 MHz, effective radiated power: <20 dBm
- Environmental conditions for use: +10°C +40°C; RH ≤85%
- Environmental storage conditions: +10°C +50°C; RH ≤85%

The scale calculates the body composition for individuals aged 12 to 99 measuring between 90 and 220 cm tall.

RECOMMENDATIONS FOR USE

- Always weigh yourself on the same scale at the same time, preferably when you wake up and while naked.
- It is important to weigh yourself with dry, bare feet positioned correctly on the electrodes (see figure). Also avoid contact between the legs so the scale operates correctly.
- Wait about 15 minutes after rising so water can distribute itself through your body.
- Scale precision could vary in the following cases:
 - a) high level of alcohol
 - b) high level of caffeine or drugs
 - c) after heavy meals
 - d) after rigorous sports activities
 - e) during illness
 - f) when pregnant
 - g) when dehydrated
 - h) with wet or unclean feet
 - i) swollen legs

INSTRUCTIONS FOR USE

FIRST USE

- These scales require 4 x 1.5V type AAA replaceable alkaline batteries for operation.
- To insert the batteries, open the battery compartment by sliding the cover in the direction of the arrow.
- Insert the batteries, making sure the polarity is correct. Close the cover.

UNIT SELECTION

Switch on the scales by stepping onto them and checking the unit of measurement indicated on the display (kg or lb). To change the unit of measurement, step off the scales and press the button inside the battery compartment underneath the scales.

USE

a. Measuring weight only

Body composition data is only available via the "LAICA Home Wellness" smartphone app; the scale's display only shows weight.

It can therefore be used as a simple scale in the following way:

- 1) Place the scale on a flat, sturdy surface. An uneven or disjointed floor (e.g., rug, carpet, linoleum) may compromise the accuracy of the reading.
- 2) With the scale off, step on with parallel feet and your weight equally distributed (see drawings). Do not move until the scale has measured your weight. Do not lean on anything while the scale measures your weight in order to avoid altering the readings.
- 3) Step off the scale. The display will continue to show the reading for a few seconds before automatically turning off.

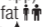
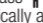
b. Operation with app: calculation of body composition

List of compatible devices:

- **iOS:** iPhone with 10 operating system.
- **For device with Android™ operating system:** with the 6.0 version (Marshmallow).

Compatible with bluetooth 4.0 and higher technology, the scale sends data on body composition (fat percentage, water, skeletal muscle, basal metabolic data and body mass index) to the smartphone to

better analyse personal weight composition. During the following operations, we recommend keeping the smartphone near the scale to keep the bluetooth connection active.

1. Activate bluetooth in the smartphone settings.
2. Install the free "LAICA Home Wellness" app on your smartphone
3. Start the App and follow the instructions to add the product and to create the new user by entering your personal data.
4. With the scales switched off, step onto the platform, taking care to position your bare feet on the ITO electrodes. The scales display:
 - a. weight
 - b. the symbol  that indicates that the connection between the scale and the smartphone is active.
5. To send the data detected by the scale to the smartphone, press the "DETECT WEIGHT" button in the app.
6. At this point the following data is displayed on your smartphone:
 - weight
 - weight goal (if set in the app)
 - the percentage of body fat 
 - the percentage of water 
 - basal metabolism 
 - body mass index (BMI) 
 - the percentage of muscle mass 
7. The scale switches off automatically after approximately 10 seconds of non-use.

Compare the fat percentage reading with the parameters indicated in the "Body fat table" if outside normal values, consult your doctor. Evaluate the water percentage, as indicated in the "water %" paragraph, as well as muscle percentage as indicated in the "muscle %" paragraph.

- When weighing, check that the app is on and displayed on your smartphone to ensure instant transfer of data about your body composition.
- Before taking a new reading, always make sure that the bluetooth connection on your smartphone is active.
- For Android™ devices it is necessary to activate the localisation services.
- Before taking a new weight measurement, make sure the correct user is selected on your smartphone.
- Each time the scale is moved and then returned to its resting surface, the first weight must not be considered due to reorganization of the electronic board with the loading cells. Consider the second and successive weights, as in certain cases the first one could be altered for the reason mentioned above.

REPLACING BATTERIES

When the "LO" or battery symbol appears on the display, replace the batteries with the same type.

1. Open the battery compartment by sliding the cover in the direction of the arrow.
2. Remove the worn batteries and dispose of them according to the instructions in the paragraph "Disposal Procedure".
3. Insert the new batteries according to the pole markings.
4. Close the battery compartment.

MAINTENANCE

- Store the appliance in a cool, dry area.
- Store the device in a horizontal position
- Only clean the scale with a soft, dry cloth.

- Never use chemicals or abrasives.

TROUBLESHOOTING

Problem	Possible cause	Solution
The scale doesn't turn on.	You have not stepped onto the scale properly.	Step onto the scale with your feet in a parallel position. Attention: only turn on after stepping on the scale.
	The batteries are worn.	Replace the batteries.
"Lo" or the battery symbol is shown on the display.	The batteries are low.	Replace them.
The display shows "Err".	Weight greater than the scale capacity.	Step off the scale immediately so as not to damage it and check the maximum capacity.
	The bluetooth function has not been enabled in the smartphone settings.	Enable bluetooth on your smartphone.
	The smartphone went in standby during weighing.	Keep the app active on your smartphone during weighing to check that data is sent instantly.
The "LAICA Home Wellness" app doesn't show any data.	The "DETECT WEIGHT" button has not been pressed in the app.	Press the "DETECT WEIGHT" button in the app to send the data from the scale to the app.
	The "LAICA Home Wellness" app only displays the body weight.	You did not step on the scale with bare feet.
The skin on your feet is too dry.		Slightly dampen your feet.

All weights are approximate having been rounded up or down to the nearest whole number. We guarantee accuracy within +/-1% + 1 division.

DISPOSAL PROCEDURE

The symbol on the bottom of the device indicates the separated collection of electric and electronic equipment (**Dir. 2012/19/Eu-WEEE**).

At the end of life of the device, do not dispose it as mixed solid municipal waste, but dispose it referring to a specific collection centre located in your area or returning it to the distributor, when buying a new device of the same type to be used with the same functions. If the appliance to be disposed of is less than 25 cm, it can be returned to a retail location that is over 400 m² without having to purchase a new, similar device.

This procedure of separated collection of electric and electronic devices is carried out forecasting a European environmental policy aiming at safeguarding, protecting and improving environment quality, as well as avoiding potential effects on human health due to the presence of hazardous substances in such equipment or to an improper use of the same or of parts of the same. Caution! The wrong disposal of electric and electronic equipment may involve sanctions.

To correctly dispose of batteries (**Dir. 2013/56/Eu**) **do not throw them into household waste** but rather dispose of them as special waste at specialised recycling centres.

For more information about disposal of used batteries, contact the store where you purchased the equipment containing the batteries, the municipality or the local waste disposal service.

This product is meant for domestic use.

The conformity, certified by the CE-marking on the device, refers to the Directive 2014/30/Eu on electromagnetic compatibility.

This product is an electronic device that was tested to assure, according to the current technical knowledge, that it does not interfere with other nearby located equipment (electromagnetic compatibility) and that it is safe if used according to the advices given in the instructions for use. In case of operation anomalies, do not keep using the device and, if necessary, refer to the manufacturer directly.

The manufacturer, Laica S.p.A., declares that the device is compliant with Directive 2014/53/Eu. The complete statement of compliance is available at the following Internet site: www.laica.it.

WARRANTY

This equipment is covered by a 2-year warranty starting from the purchase date, which has to be proven by means of **the stamp or signature of the dealer and by the fiscal payment receipt that shall be kept enclosed to this document.**

This warranty period complies with the laws in force and applies only in case the consumer is an individual. Laica products are designed for domestic use only; therefore, their use for commercial purposes is forbidden. The warranty covers only production defects and does not apply, if the damage is caused by accidental events, misuse, negligence or improper use of the product. Use the supplied accessories only; the use of other accessories invalidate the warranty cover. Do not open the equipment for any reason whatsoever; the opening or the tampering of the equipment, definitely invalidate the warranty. Warranty does not apply to parts subject to tear and wear along the use, as well as to batteries, when they are included. Warranty decays after 2 years from the purchase date.

In this case, technical service interventions will be carried out against payment. Information on technical service interventions, under warranty terms or against payment, may be requested by writing at info@laica.com. Repair or replacement of products covered by the warranty terms are free of charge. In case of faults, refer to the dealer; DO NOT send anything directly to LAICA.

All interventions covered by warranty (including product replacement or the replacement of any of its parts) shall not be meant to extend the duration of the original warranty period of the replaced product. The manufacturer cannot be made liable for any damages that may be caused, directly or indirectly, to people, things and pets resulting from the inobservance of the provisions contained in the suitable user manual, especially with reference to the instructions related to installation, use and maintenance of the equipment. Being constantly involved in improving its own products, Laica reserves the right to modify its product, totally or partially, with reference to any production need, without that this involves any responsibility whatsoever by Laica or its dealers.

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